


RUBBER THRESHOLD/CARGO WEDGE™ RAMP



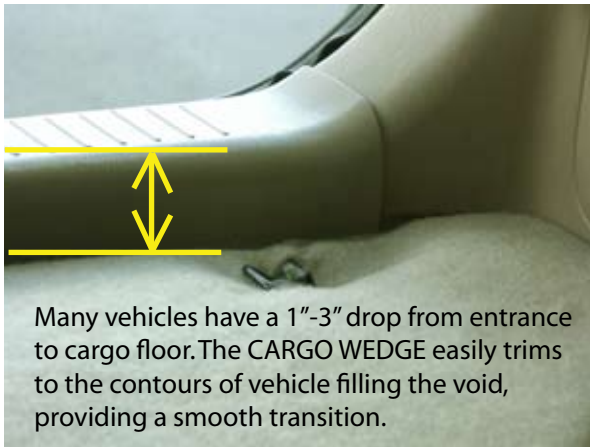

Many vehicles have a 1"-3" drop from entrance to cargo floor. The CARGO WEDGE easily trims to the contours of vehicle filling the void, providing a smooth transition.

Shown without the CARGO WEDGE



CARGO WEDGE shown with an Advantage Series SUITCASE ramp with the TLE option

The rubber CARGO WEDGE™ is the ideal solution when recessed cargo areas leave you in a rut. Without altering the vehicle, the CARGO WEDGE compensates for the difference between the rear door threshold and the lower-level cargo area, allowing mobility equipment to be loaded/unloaded with a smooth transition. The CARGO WEDGE, when used in conjunction with the SUITCASE®, or TRIFOLD® Advantage Series® Ramp, or SUITCASE® Signature Series® Ramp, provides the perfect transport system for mobility access. Slip-resistant and made of recycled tires.

SPECIFICATIONS	THR 250-1
Made of recycled rubber	•
Non-skid surface	•
Usable size (H x L x W)	2½" x 24" x 48" 
Weight (lbs.) *	40
Weight capacity (lbs.)	850 
For wheelchairs & scooters	•
Usage guidelines	2½" rise


* Weights are approximate and may vary. Consult your equipment's owner guide for proper degree of incline. Never exceed its recommendations.

Step 1. Mark trim guidelines.



Step 2. Trim with a utility knife or power saw.

Step 3. Position custom trimmed CARGO WEDGE into recessed area.

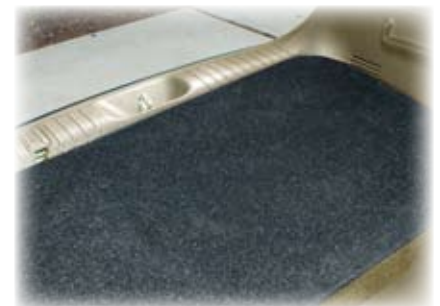

Bariatric weight capacity = 800 lbs. or greater



Bariatric width = 36" or greater

