

INSTRUCTIONS FOR INSTALLATION AND USE

BEFORE YOU BEGIN:

1. Maximum weight limit is 850 pounds.
2. Read and follow all instructions before use. Do not use if ramp is damaged or unstable.
3. For wheelchairs and scooters only; ramp is not intended for commercial use.
4. **WARNING:** Use ramp only with a qualified helper and always use your lap belt.
5. **WARNING:** It is important that you refer to your equipment's (wheelchair or scooter) Owner Guide for the proper degree of incline/decline and chair direction before using ramp. Never exceed its recommendations.

FITTING A RUBBER THRESHOLD RAMP:

1. Ramp can be easily trimmed to accommodate lower heights and narrower widths and, if necessary, notched to suit the doorjamb configuration.
2. If door threshold is equal to the height of the ramp, use Threshold Ramp as is.
3. If door threshold is less than the height of the ramp, trim ramp to the desired height as shown in DIAGRAM 1 (serrations in the bottom of the ramp can be used as a guide). **WARNING:** Use caution when operating any non-manual saw as serious injury may result due to unexpected binding or pinching of rubber ramp material. Always follow tool manufacturer's safety guidelines.

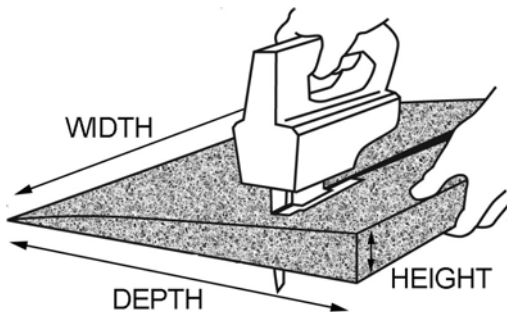


DIAGRAM 1

SECURING THE RAMP:

It is recommended that the Rubber Threshold ramp be secured to the surface it is laying on.

1. Attaching to wood:
 - a. Place ramp in desired location and ensure that surface is free of dirt and debris.
 - b. Secure ramp using supplied Phillips head screws (two each). Space screws evenly (see DIAGRAM 2 for placement) and drive until head is ¼" below the ramp surface.
2. Attaching to concrete:
 - a. Place ramp in desired location and ensure that surface is free of dirt and debris.
 - b. Secure ramp using supplied Phillips head screws and concrete anchors (two each).
 - c. Place a cardboard template under the ramp, and then install screws in locations shown in DIAGRAM 2. Drive screws until they penetrate the cardboard template (top of screw will be roughly ¼" below the ramp surface).
 - d. Remove ramp, but **DO NOT REMOVE SCREWS**. Using the holes made in the cardboard as a guide, drill anchor holes into concrete using a 5/16" masonry bit.
 - e. Remove template, then insert anchors into concrete. Drive each screw into anchor until the screw head is ¼" below the ramp surface.

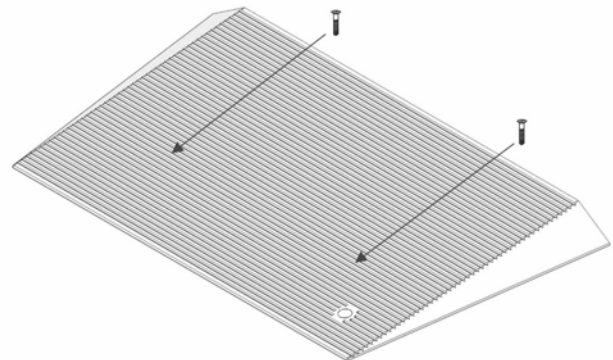


DIAGRAM 2

CLEANING AND MAINTENANCE:

1. At all times, keep ramp clear of snow, ice, dirt, leaves, and other debris that may accumulate on the surface.
2. Clean ramp with mild soap and warm water. Rinse well and use extra caution when ramp surface is wet.

